MEDICAL PROGRAMM

2020/2021

Sicher. Sauber. **ALPRO.**

ALPRO MEDICAL GMBH

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 For more than 30 years, **ALPRO** is one of the most innovative companies in the infection control sector. **ALPRO** stands out due to first class and environment-friendly products and a maximum of reliability. Furthermore, **ALPRO** is one of the world's leading specialists in the cleaning, maintenance and disinfection of dental suction systems with and without amalgam separator (AlproJet-D, AlproJet-DD, AlproJet-W, AlproCleaner).

ALPRO attaches great importance to environmental friendliness, aldehyde-free products are therefore a matter of course for us. These include the practice proven products MinutenSpray-classic/ MinutenWipes variants for quick surface disinfection, as well as the universally applicable products BIB forte eco (burr and instrument bath) for the disinfection and the cleaning of all instruments as well as PlastiSept eco/Wipes variants, the current gentle alcohol-, aldehyde- and QAC-free disinfectant for all plastic and metal surfaces.

ALPRO can also show excellent results in the research and development of products for the removal of biofilm in dental unit water lines and the decontamination of procedural water. With BRS® (Biofilm-Removing-System) and Alpron/Bilpron the requirements of EN ISO 16954:2015 are met and drinking water quality in dental units is achieved. In order to maintain this standard, which is necessary for the dental practice, we provide two different systems for the decontamination of procedural water, Weekend-System II and ALPRO-BCS (BottleCareSystem), for the connection to the dental unit.

Another highlight of our product range is the WL-line, cleaning and disinfection solutions for manual processing of handpieces and turbines. These device-friendly solutions comply with the highest cleaning requirements (EN ISO 15883) and disinfection requirements and were furthermore tested and approved by several manufacturers.

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- PZN (für 300 ml): 11857502
- PZN (für 1 L): 03645335

 Rinsing solution ready for use for application in mouth rinsing devices and ultrasonic devices. Alprox serves for cleaning and care of the mouth cavity as well as of the gingival pockets. Alprox reduces the vitality of periodontitis bacteria, removes plaque and protects against tartar. Alprox does not lead to discolourations of the teeth. Alprox has no negative effect on the action of detergents and disinfectants in the aspiration system.

Active ingredient basis

modern combination of biguanide and tosylchloramide

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Use biocidal products carefully. Always read the label and product information before use.

 Quick acting, ready for use alcoholic rub-in-product for the hygienic hand disinfection.

For hygienic hand disinfection, rub 3 – 5 ml AlproSept-HDE/HDE gel and keep moist for 30 seconds.

Efficacy/Application

- **bactericidal** incl. MRSA and suitable for hospitalism prophylaxis
- **yeasticidal** (*C. albicans*)
- **virucidal, active against all enveloped "blood-borne" viruses**
e.g. corona viruses as well as non-enveloped adeno viruses
- proved according to EN 1040, EN 1275, EN 1500 and EN 14476
- hygienic hand disinfection 30 sec.

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Application notes

Standard rubbing method for surgical hand disinfection according to EN 12791

For the **surgical hand disinfection** take the hand disinfectant into your cupped, dry hands and rub for **3 minutes** up to the elbows following the listed steps.

Pay attention to keep the hands moist over the entire rubbing time.

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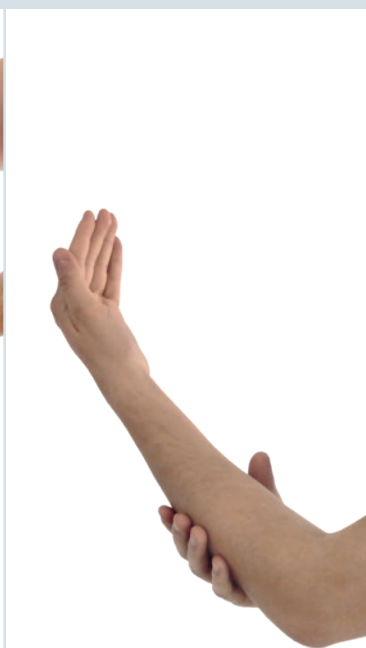
Step 1

Apply hand disinfectant on dry hands. Use elbow to operate the dispenser.

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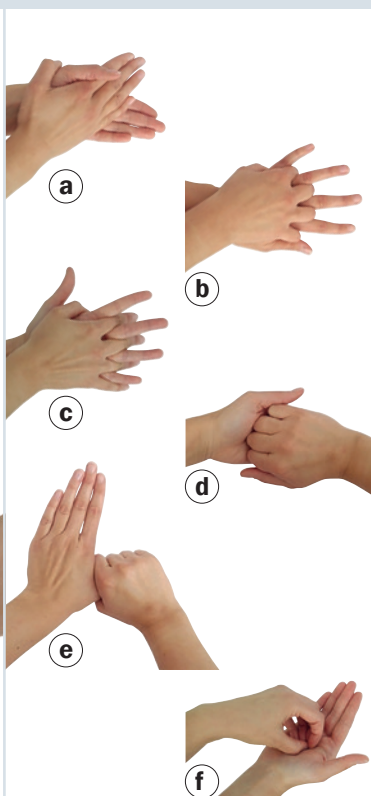
Step 2

Rub forearms up to the elbows with hand disinfectant for **at least 3 minutes**.

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Step 3

Disinfect hands. Follow rubbing method for hygienic hand disinfection.

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Step 4

Keep hands above elbow level during and after disinfection. Hands must be air-dried before wearing surgical gloves.

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Application notes

Standard rubbing method for hygienic hand disinfection according to EN 1500

For the **hygienic hand disinfection** take the hand disinfectant into your cupped, dry hands and rub for **30 seconds** up to the wrists following the listed steps. The moves of each step have to be repeated five times. After finishing step 6 repeat the single steps until the indicated rubbing time has been attained.

Pay attention to keep the hands moist over the entire rubbing time.

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Step 1 (approx. 5 seconds)
rub the hand palms together, additionally the wrists

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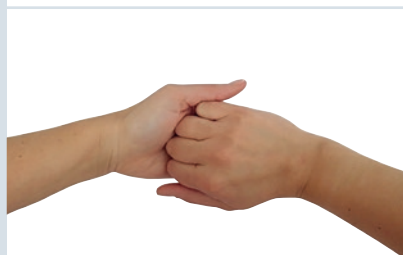
Step 2 (approx. 5 seconds)
rub right hand palm against the back of the left hand and vice versa

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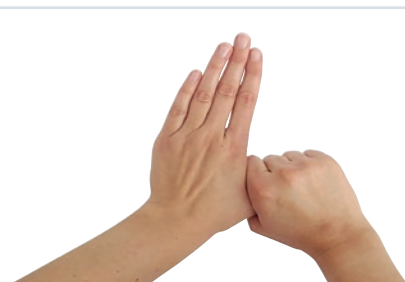
Step 3 (approx. 5 seconds)
palm to palm with spread fingers

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Step 4 (approx. 5 seconds)
rub the outsides of the folded fingers against the opposite palm

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Step 5 (approx. 5 seconds)
rub the left thumb in circles in the closed right hand and vice versa

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Step 6 (approx. 5 seconds)
move the closed fingertips in circles in the right palm and vice versa

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 Mild washing lotion for frequent hand washing. Cleans the skin particularly gently. Contains skin protection substances that maintain the natural acid mantle of the skin. CREMANA®-wash is pH-neutral and has a reserved scent.

Field of application

Ideal for the use in all medical fields that require frequent hand washing, e. g. medical and dental practices, hospitals, etc.

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 Cream for intensive skin protection and care. CREMANA® Creme-Lotion is easy to rub in, absorbs quickly and does not grease. Suitable for all skin types.

Application

After the cleaning/decontamination of the hands (breaks/end of work) put a cherry stone sized quantity on the back of the hand and rub in thoroughly.

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 Greasing skin protection and barrier cream (W/O) free of silicone, for application at work in case of frequent contact with water based products, like cleaning and disinfection agents. Ideal for skin protection when wearing gloves.

Application

Apply on washed and dried hands. For the sufficient protection of the hands put a cherry stone sized quantity on the back of the hand and disperse to the palm of the hand.

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 Alcohol-free wet wipes for the non protein fixing cleaning and decontamination of surfaces of medical devices and intact skin. The bacteriostatic and fungistatic acting CleanWipes in „cotton touch“ quality, remove easily and carefully organic and inorganic soiling from medical devices, hands and intact skin. CleanWipes are dermatologically tested.

Active ingredient basis

EDTA, biguanides in glycolic-aqueous solution

Efficacy/Certification

- cleaning/decontaminating
- dermatologically approved

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 Alcoholic solution ready for use for the quick cleaning and disinfection of surfaces of medical devices such as turbines, hand pieces and contra-angles, unit grips, suspension tables and medical equipment. Approved amongst others by Dentsply Sirona, Planmeca. Only for use on alcohol compatible surfaces.

Contact times (undiluted)

- | | |
|--|---------|
| • bactericidal (incl. TBC, MRSA) | 1 min. |
| • yeasticidal | 1 min. |
| • virus inactivating limited spectrum of virucidal activity according to EN 16777: | |
| • all enveloped „blood-borne“ viruses | 30 sec. |
| • as well as non-enveloped viruses (see efficacy) | 2 min. |

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Efficacy/Certification

- **bactericidal** incl. TBC, MRSA and suitable for hospitalism prophylaxis
- **yeasticidal**
- **virucidal** limited spectrum of virucidal activity according to EN 16777: non-enveloped adeno-, noro viruses as well as all enveloped „blood-borne“ viruses, such as HBV, HCV, HIV and influenza
- with remanence effect (inhibition of recontamination)
- certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology), meets the DGHM requirements and EN-standards for surface disinfection

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-  Alcoholic wet wipes for the quick cleaning and disinfection of surfaces of medical devices such as turbines, hand pieces and contra-angles, unit grips, suspension tables and medical equipment.
- free of aldehydes and phenols, VAH certified, ready for use, lint-free

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Alcoholic wet wipes for the quick cleaning and disinfection of surfaces of medical devices such as turbines, hand pieces and contra-angles, unit grips, suspension tables and medical equipment.

- free of aldehydes and phenols, VAH certified, ready for use, lint-free

Active ingredient basis

Quick-acting aldehyde-free and phenol-free disinfectant based on a combination of alcohols, QACs and alkylamines.

Efficacy/Certification

Test criteria: dirty conditions, room temperature:

- **bactericidal** (EN 13727, EN 16615) in 2 minutes
- **tuberculocidal** (EN 14348) in 1 minute
- **MRSA** (suspension test) in 1 minute
- **yeasticidal** (EN 13624, EN 16615) in 2 minutes

Test criteria: clean conditions, room temperature:*

- **virucidal** (EN 14476) against non enveloped adeno- and noro viruses as well as all enveloped viruses in 2 minutes

Test criteria: without soiling, room temperature:*

- **virucidal** (suspension test) against enveloped viruses, e.g. HBV, HCV, HIV and Influenza in 30 seconds

We recommend a contact time of 2 minutes.

- * Pre-cleaning: in accordance with the recommendation of the RKI (Robert Koch Institute), heavily soiled surfaces should be pre-cleaned.

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- Application only on alcohol compatible surfaces by qualified medical personnel!

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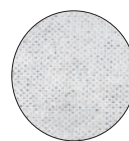
 Alcohol-free solution ready for use for the cleaning and disinfection of sensitive surfaces of medical devices such as medical and dental units (incl. upholstery and suspension tables) and X-ray machines particularly with acrylic glass or metal surfaces.

Contact times (undiluted)

- bactericidal 1 min.
- yeasticidal 1 min.
- TBC 5 min.
- virus inactivating according to EN 16777 active against:
 - all enveloped incl. „blood-borne“ viruses incl. corona viruses (EN 14476) 5 min.

Active ingredient basis

Current alcohol-free, aldehyde-free and QAC-free formulation based on sec. and tert. alkylamines.

Efficacy/Certification

- **bactericidal** incl. TBC and suitable for hospitalism prophylaxis
- **yeasticidal**
- **virucidal** according to EN 16777, active against all enveloped incl. „blood-borne“ viruses, such as BVDV, corona viruses (EN 14476), HBV, HCV, HIV, influenza, vaccinia as well as non-enveloped noro viruses (120 min.)
- certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology), surface disinfection with mechanics

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 Alcohol-free wet wipes for the cleaning and disinfection of sensitive surfaces of medical devices such as medical and dental units (incl. upholstery and suspension tables) and X-ray machines particularly with acrylic glass or metal surfaces.

- free of aldehydes and phenols, VAH certified, ready for use, lint-free

Application

Particularly suitable for surfaces which are sensitive to alcohol.

Active ingredient basis

Alcohol-free, aldehyde-free and phenol-free formulation based on sec. and tert. alkylamines.

Efficacy in 5 minutes

Test criteria: clean conditions, room temperature*:

- **bactericidal** (EN 13727, EN 16615)
- **yeasticidal** (EN 13624, EN 16615)
- **tuberculocidal** (carrier test) + EN 16615
- **virucidal** (EN 14476, EN 16777) against enveloped viruses, e.g. corona viruses, HBV, HCV, HIV and influenza

Note in case of an epidemic:

non enveloped noro viruses in 120 minutes (EN 16777)

* Pre-cleaning: in accordance with the recommendation of the RKI (Robert Koch Institute), heavily soiled surfaces should be pre-cleaned.

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 Large dry wipes „cotton touch“ quality in the MaxiWipes reel-Box for the cleaning and disinfection of medical devices, especially for larger surfaces, in combination with MinutenSpray-classic or PlastiSept eco.

Contact times – soaked with PlastiSept eco

- bactericidal 5 min.
- yeasticidal 5 min.
- virucidal active against all enveloped viruses according to EN 16777:
 - enveloped viruses incl. corona viruses (EN 14476) 5 min.
- TBC 5 min.

Contact times – soaked with MinutenSpray-classic

- bactericidal (incl. TBC, MRSA) 2 min.
- yeasticidal 1 min.
- virucidal limited spectrum of virucidal activity according to EN 16777:
 - enveloped viruses 30 sec.
 - non-enveloped viruses 2 min.

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Field of application, soaked with PlastiSept eco

Especially suited for surfaces which are sensitive to alcohol such as treatment units (e. g. upholstery, head rests), suspension tables, in the X-ray area (e. g. chin rests at OPG), and surfaces of acrylic glass.

Field of application, soaked with MinutenSpray-classic

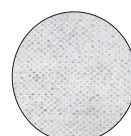
Suited for surfaces which are compatible with alcohol if quick drying and a comprehensive disinfection are required.

Active ingredient basis/Efficacy

- soaked with PlastiSept eco s. p. 20
- soaked with MinutenSpray-classic s. p. 15

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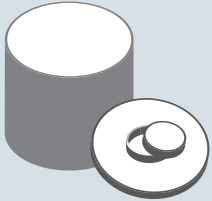
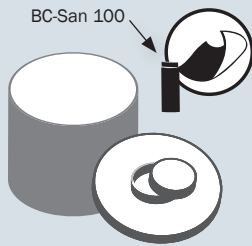
Step 1

Dispose of solution and wipes beyond expiry date.

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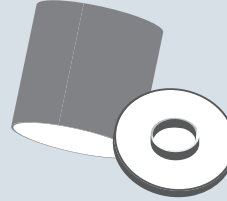
Step 2

Wipe thoroughly the inside of the empty box and the lid for processing with a cloth soaked with BC-San 100.

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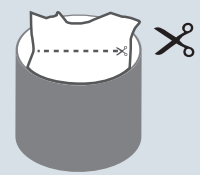
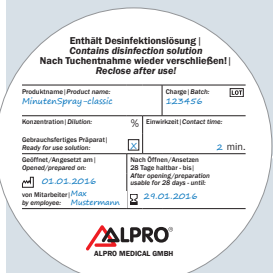
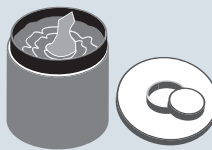
Step 4

Place the bag into the dispenser box and cut it open along the dividing line.

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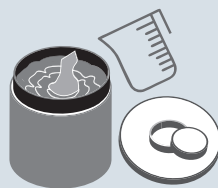
Step 2 - Filling

Put the unpacked reel with bag into the box. Pull out a small piece of the innermost wipe from the middle.

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Step 3 - Soaking

Then slowly pour the ready for use solution (MinutenSpray-classic or PlastiSept eco) spirally from the middle outwards over the reel. Thus you ensure a complete wetting.

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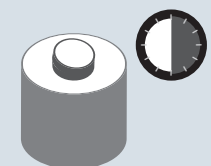
Step 4 - Stick through

Now pull the innermost wipe of the reel through the star-shaped tear-off slot of the lid and throw it away. Put the lid on the dispenser and close until it locks.

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Step 5 - Soaking time

For complete soaking it is sufficient to let the wipes rest for 30 minutes.

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 Liquid concentrate free of aldehydes and phenols with comprehensive microbicidal efficacy for the cleaning and disinfection of surfaces of medical devices.

Contact times

- **General hygiene and prophylaxis**
bactericidal, fungicidal and virucidal against enveloped viruses (s. efficacy) 1.0 % 60 min.
virucidal against enveloped viruses (s. efficacy) 2.0 % 15 min.
- **Critical areas (e. g. operating room)**
bactericidal, fungicidal and virucidal (**against enveloped and non-enveloped „blood-borne“ viruses**) (see efficacy) 4.0 % 60 min.
5.0 % 15 min.

Active ingredient basis

Contemporary combination based on QAC, tensides and complexing agents.

Efficacy/Certification

- **bactericidal** incl. MRSA and suitable for hospitalism prophylaxis
- **fungicidal** (A. niger, C. albicans)
- **virucidal** according to EN 14476 all enveloped and non-enveloped viruses incl. polio
- **sporizidal** against C. difficile (EN 13704)

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 Alkaline-enzymatic cleaning granulate for the pre-cleaning and pre-disinfection for the processing of medical and dental instruments, rotating precision instruments and endoscopes. Useable by soaking bath method as well as in ultrasonic devices and washer-disinfectors.

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Phosphate, Tenside, Parabene, Entschäumer, Stabilisatoren, Enzyme (gecoated).

Active ingredient basis

Phosphates, tensides, parabens, defoaming agents, stabilizers, coated enzymes.

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Efficacy

- high cleaning performance (already at 0.5 % application)
- removes blood and other protein-containing body fluids as well as biofilm

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Spectrum of activity (in soaking bath)

- cleaning (ISO/TC 198/WG 13) 0.5 % at 45 °C
- virucidal (EN 16777; dirty conditions) against non enveloped adeno-, noro- and parvo viruses as well as against enveloped viruses, e.g. HBV, HCV, HIV and Influenza 1.0 % in 60 minutes at 30 °C

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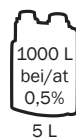
Manual cleaning and disinfection of instruments

CE 0123

BIB forte eco

 Liquid concentrate free of aldehydes, phenols and QAC for the non protein fixing cleaning and disinfection of medical and dental instruments, rotating precision instruments and endoscopes. Tested on instruments of company Komet, furthermore approved and recommended by Acurata, Busch & Co, Dentsply Maillefer, VDW, Helmut Zepf, Hager & Meisinger and Oridima.

Spectrum of activity (at high protein load):

- **bactericidal incl. TBC and MRSA, yeasticidal (according to EN)**

economical	0.5 %	60 minutes
	1.0 %	30 minutes
quick	3.0 %	10 minutes in ultrasonic bath
	4.0 %	5 minutes in ultrasonic bath
- **virucidal activity: enveloped viruses such as HBV, HCV, HIV, Herpes simplex, Influenza (H1N1 + H5N1), BVDV, Vaccinia**

	0.5 %	60 minutes
	1.0 %	10 minutes
- **virucidal activity: enveloped viruses + adeno- and noro viruses (EN 14476)**

	3.0 %	10 minutes in ultrasonic bath (at 40 °C)
--	-------	--
- **virucidal activity: non enveloped adeno-, noro and polio viruses (EN 14476)**

	3.0 %	10 minutes in ultrasonic bath (at 55 °C)
--	-------	--
- **according to EN 16777**

	0.5 %	60 minutes
--	-------	------------

after pre-cleaning with AlproZyme

Durability in use

7 days at normal contamination.

Material compatibility

The extremely high corrosion-inhibiting properties could be improved by the QAC-free formulation, thus instruments sensitive to corrosion get optimal protection. Even with occasional exceeding of contact time BIB forte eco is absolutely compatible with the materials.

Active ingredient basis

Modern combination of alkylamine and biguanide.

Efficacy

- **bactericidal** (incl. TBC and suitable for hospitalism prophylaxis)
- **yeasticidal**
- **virucidal** limited spectrum of virucidal activity according to EN 14476 against enveloped viruses e. g. HBV, HCV, HIV, Influenza as well as non-enveloped viruses at 40 °C
- **virucidal according to EN 16777** after the pre-cleaning with AlproZyme incl. parvoviruses
- CJD-prophylaxis, meets the requirements of the RKI (pH ≥ 10)
- VAH/DGHM listed – instrument disinfection at high organic load

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 Non protein fixing solution ready for use for the pre-cleaning of medical instruments. Dissolves residues of blood and other body fluids from medical devices (internal and external surfaces).
Approved amongst others by Planmeca and EMS.

Active ingredient basis

Complexing agent (EDTA), parabens, biguanides in aqueous solution.

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 Solution ready for use for the non protein fixing cleaning (internal and external surfaces) prior to disinfection/sterilization of medical and dental hollow part instruments such as turbines, hand pieces and contra-angles, internally cooled instruments and endoscopes. WL-clean is a medical device of class I. The proof of suitability for cleaning has been given according to the series of standards EN ISO 15883 and according to the specifications of the German Society for Sterile Goods Supply (DGSV) – and has been confirmed by expert's reports. Tested and approved amongst others by KaVo, NSK, SciCan, W+H, Morita, Sirona.

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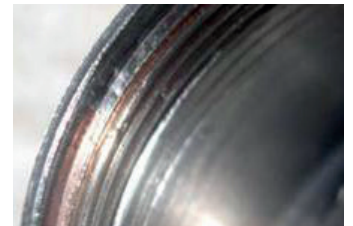
Efficacy

- complexing
- emulsifying
- not protein fixing
- tested under practical conditions with soiling test according to ISO/TS 15883-5

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 Solution ready for use for the disinfection (internal and external surfaces) of medical and dental hollow part instruments such as turbines, hand pieces and contra-angles after the non protein fixing pre-cleaning with WL-clean.

WL-cid is a medical device of class IIb. The disinfection capacity under simulated practical conditions according to VAH/DGHM has been confirmed by an expert's report.

Active ingredient basis

Aldehyde-free and phenol-free disinfection solution based on a combination of alcohols, QACs and alkylamines.

Efficacy/Certification

- **bactericidal**
- **mycobactericidal** (*Mycobacterium terrae* and *Mycobacterium avium*)
- **fungicidal**
- **virucidal** according to EN 14476 and EN 16777 against non-enveloped polio-, adeno-, noro- and polyoma viruses (SV 40) as well as all enveloped viruses, such as HBV, HCV, HIV and influenza.
- **active solution certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology)**

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Important note:

For the correct reprocessing, please refer to our latest SOP.

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- WL-cid**
- REF 4141** Kombipackung/combination set
WL-cid + WL-dry*
- REF 4140** Klinikpackung/clinic package
4 x 500 ml WL-cid*

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 Medically pure drying spray to support the cleaning after the cleaning and/or disinfection of the inner surfaces of medical and dental hollow part instruments (turbines, hand pieces and contra-angles, internally cooled instruments and endoscopes). WL-dry is a medical device of class I.

WL-dry contains: 1,3,3,3-Tetrafluorprop-1-ene
(R1234ze; CAS-no.: 29118-24-9)

Field of application

Removing of residues of cleaning and disinfection agents and maintenance products and drying of spray channels and inner surfaces.

Content

100 % chemical inert propellant, meets the hygienic requirements on „medical air“ for the processing of medical hollow part instruments.

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 The compressed air gun WL-Blow with integrated air flow limiter is used to blow out, to dry and to support the cleaning of hollow part instruments and transmission instruments after cleaning and disinfection with WL-clean/WL-cid¹. WL-Blow is intended for use in the processing room and can be connected to the existing oil-free dry air compressor!²

¹ Please note manufacturers' approvals!

² Connection of a bacterial filter is recommended!

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 The rinsing device „WaterJet“ with integrated water flow limiter is used to pre-rinse and facilitates the cleaning before the further processing of outer and inner surfaces of dental transmission instruments and general instruments. The rinsing device “WaterJet” is designed for the use in the preparation room and can be connected to the drinking water line.

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*Adapter for **all** contra-angles and hand pieces with ISO-interface for complete internal cleaning/disinfection incl. head of instrument.*

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Adapter for turbines with Sirona interface for complete internal cleaning/disinfection incl. head of turbine.

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Adapter for turbines with Multiflex-(KaVo/SciCan) interface for complete internal cleaning/disinfection incl. head of turbine.

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(WL-Adapter 02/G, 04/G) inkl. je 1 Set Ersatz-O-Ringe.

(WL-adapter 02/G, 04/G) incl. 1 replacement set of O-rings each.

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Adapter for turbines with Bien Air-interface for complete internal cleaning/disinfection incl. head of turbine.

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Adapter for turbines with Roto-Quick-interface (W&H) for complete internal cleaning/disinfection incl. head of turbine.

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Adapter for longshank contra-angles T1 Classic (Sirona) for complete internal cleaning/disinfection incl. head of instrument.

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Adapter for turbines with NSK-interface for complete internal cleaning/disinfection incl. head of turbine.

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Universal adapter with Luer connector for transmission instruments (ISO 9168) for bringing-in of cleaning and maintenance products (e. g. WL-clean, WL-cid, maintenance oil)

and for blowing-out of with WL-dry/WL-Blow. Also suitable for blowing-out by using the multi-function syringe and to bring in the cleaning solution WL-clean via Luer connection e. g. into other hollow part instruments.

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Adapter for turbines with Morita-interface for complete internal cleaning/disinfection incl. head of turbine.

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Adapter for ACTEON-AIR N GO Airpolisher Supra/Perio nozzle for complete internal cleaning/disinfection incl. head of instrument.

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Adapter for KaVo MF handpiece cannulas for complete internal cleaning/disinfection.

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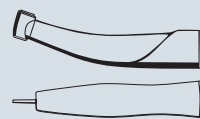
Adapter only for spray water and spray air channels of turbines with Sirona-interface.

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Adapter only for spray water and spray air channels for turbines with Multiflex-(KaVo/SciCan) interface.

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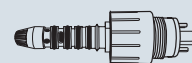
Rinsing adapter for ultrasonic tips/sonic tips M 3 internal thread from EMS, Mectron, KaVo, Komet.

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Rinsing adapter for ultrasonic tips M 3/a internal thread from Satelec, NSK.

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Rinsing adapter for ultrasonic tips/sonic tips in the ultrasonic surgery external thread from Sirona (Siroprep and Siroimplant).

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Rinsing adapter for ultrasonic tips in the ultrasonic surgery internal thread from Acteon, (2nd generation) e. g. for Piezotome.

WL-Adapter U-1 + U-1a Kombi* (Edelstahl) WL-Adapter 1-U + U-1a combi* (stainless steel)

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Combined adapter to clean and disinfect surgical contra-angles and handpieces externally cooled with the WL-series. Consisting of the U-1 rinsing adapter for ultrasonic tips and U-1a screw attachment for hose plug connection.

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* Please note manufacturers approvals!

Manuelle Instrumentenreinigung und -desinfektion Manual cleaning and disinfection of instruments

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 Solution for the virucidal final disinfection of surfaces of medical devices, such as light guides, polymerisation lamps, heads of intraoral cameras/scanners, as well as for the disinfection/processing of wipe dispenser boxes and pressure bottles (1 - 1.5 L) of stand-alone procedural water supply systems such as the ALPRO-BCS (BottleCareSystem).

Virucidal final disinfection:

After pre-cleaning and general disinfection (e. g. with PlastiSept eco Wipes/MinutenWipes) soak a suitable cellulose wipe with undiluted BC-San 100, thoroughly wipe off the surfaces to be disinfected and let it take effect for at least 5 minutes.

Disinfection effect tested with parvoviruses and B. subtilis spores.

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 *Mild alkaline liquid cleaning concentrate for the application in automatic and semi-automatic dosing systems of washer-disinfectors, for the automatic processing of medical devices.*

Field of application

- alkaline cleaner (pH 10) for very effective removal of blood and protein residues at all temperature programs
- especially gentle to materials
- protects instruments made of steel and stainless steel from corrosion
- suitable in combination with TR-Neutralizer also in washer-disinfectors as e. g. Miele thermodesinfectors. Approved by Melag.

Active ingredient basis

Optimized cleaning concentrate based on mild alkaline tensides, QAC, complexing agents and phosphonates.

Dosage

dosage: 0.3 % - 0.5 %

ultrasonic device: 5 - 10 ml/L

Fill the concentrate into the door dosage system for the cleaner and close the flap. For automatic dosage systems connect the dosage lance with machine-compatible can and if necessary adjust the dosage quantity.

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5 L can, compatible for direct connection with dosage lance*

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Wirksamkeit

- neutralisierend
- korrosionsverhindernd

 Liquid concentrate for the neutralization after the alkaline cleaning in washer-disinfectors.
Approved amongst others for Miele Professional G 7735, G 7781, G 7881, SIROCLEAN, MELAtherm.

Field of application

Neutralization component based on phosphoric acid for use in washers and washer-disinfectors during thermal preparation in the Vario TD program or RKI procedure for neutralization after the alkaline cleaning.

Also suitable for basic cleaning of dental instruments as immersion bath (see instructions of the manufacturer).

Active ingredient basis

Universally applicable neutralizing solution for all alkaline cleaning concentrates based on phosphoric acid.

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Inhalt

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 Coarse-grained special salt for water softeners in washers and washer-disinfectors.
Prevents calcium deposits in the equipment and on the wash items.

Content

highly purified NaCl, without suspending agent

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 RegClean is designed for the intensive cleaning and regeneration of washers/washer-disinfectors (chemical/chemical-thermal) and maintains the efficiency of the devices. RegClean removes powerfully bone substances, tissue and lime residues having formed in the spray arm and filter of the washer/washer-disinfector.

Active ingredient basis

Highly effective cleaning granulate for the cyclic cleaning of the inside, lines and sump pit based on sulfamic acid.

- Anwendung von RegClean erfolgt ohne Beladung.
- The application of RegClean is carried out without load.

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 Liquid concentrate for daily disinfection and simultaneous cleaning of dental aspiration systems with and without amalgam separator. Free of phenols, aldehydes, phosphates and chlorine!

Application

For reliable disinfection it is recommended to use AlproJet-D for 3 consecutive working days 1 – 2 times each day (e.g. noon, evening). For the remaining 2 working days the application with AlproJet-W is recommended.

Active ingredient basis

Highly effective aldehyde-free, phenol-free and chlorine-free cleaning- and disinfection concentrate based on QAC, tensides and alkaline carriers.

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Efficacy

- bactericidal
- yeastcidal
- VAH certified

Spectrum of activity

5 % ready-to-use solution with drinking water (room temperature)

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Material compatibility

Tested and approved for use on dental units by well-known dental equipment manufacturers (including Dentsply Sirona, Morita) with regard to material compatibility. This means that AlproJet-D can be used in the dental unit overnight and / or over the weekend.

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 Liquid concentrate with enhanced disinfecting effect for the daily cleaning and maintenance of dental aspiration systems with and without amalgam separator. Approved amongst others by Dentsply Sirona.

Application

For the reliable cleaning, disinfection and care it is recommended that AlproJet-DD is used on 3 days a week, 1 – 2 x per day (at noon and in the evening). For one application 2 L of a 2 % solution are needed. Let take effect for at least 60 min. (DVG 2012); rinse aspiration tubes and spittoon bowl thoroughly with water just before starting to work. Please follow the manufacturer's instructions.

Active ingredient basis

Highly effective aldehyde-free, phenol-free and chlorine-free cleaning concentrate with extended disinfection effect based on QAC, amines, tensides and alkaline carriers.

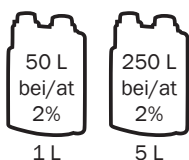
Efficacy/Certification

- **bactericidal**, EN practical test with protein load (EN 13727)
- **yeastcidal**, EN practical test with protein load (EN 13624)
- **virucidal** limited spectrum of virucidal activity according to EN 16777 against non-enveloped adeno and noro viruses as well as all enveloped „blood-borne“ viruses such as HBV, HCV, HIV and influenza
- certified by VAH (Association for Applied Hygiene)/ DGHM (German Society for Hygiene and Microbiology), meets the DGHM requirements and EN-standards for the disinfection of instruments

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Compatibility Very good compatibility with materials. Approvals from leading device manufactures.

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- AlproJet-DD**
- REF 3120 1 L Dosierflasche/dosage bottle*
 - REF 3115 5 L Kanister/can*
 - REF 3116 5 L Cubitainer*
 - REF 3125-N AlproJet-DD StarterSet*
 - REF 3085 AlproJet MixCup*

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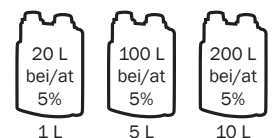
 Liquid concentrate for the weekly acid contra-cleaning and maintenance of dental aspiration systems with and without amalgam separator.
Approved amongst others by Dentsply Sirona.

Application

For the reliable cleaning and care it is recommended that AlproJet-W is used on 2 days a week, 1 – 2 x per day (at noon and in the evening) instead of AlproJet-D or AlproJet-DD. In case of high protein load AlproJet-W can also be used temporarily every day. For one application 2 L of a 5% solution are needed. Let take effect for at least 30 minutes; rinse aspiration tubes and spittoon bowl thoroughly with water just before starting to work. Please follow the manufacturer's instructions.

Active ingredient basis

Highly effective aldehyde-free, phenol-free and chlorine-free cleaning concentrate for the acid contra-cleaning based on sulfamide acid, QAC and tensides.

Efficacy

- against pseudomonas bacteria
- in combination with AlproJet-D/AlproJet-DD comprehensively cleaning and disinfecting

Compatibility

Very good compatibility with materials.
Approvals from leading device manufactures.

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Wirksamkeit

- bakterizid
- levurozid

 Liquid cleaning agent ready for use with disinfecting effect for the spittoon bowls of dental units. Approved amongst others by Dentsply Sirona.

Application

Spread AlproCleaner generously (10 – 20 ml) in the spittoon bowl and remove visible residues with a plastic brush. Rinse with clear, cold water.

Active ingredient basis

Quick-acting, alkaline cleaning solution for the direct application based on alcohol, QAC, tensides and complexing agents.

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 Solution ready for use, free of aldehydes and phenols, for the cleaning and disinfection of all kinds of dental impressions, worn dental prostheses and dental works.

PrintoSept-ID can also be used in disinfection devices for the processing of impressions, models, bite templates, auxiliaries and impression tray.

Spectrum of activity, test criteria: dirty conditions, room temperature

• bactericidal (VAH-method 15)	1 min.
• yeasticidal (VAH-method 15)	1 min.
• tuberculocidal (VAH-method 15)	5 min.
• virucidal (RKI/DVV, EN 14476) against all enveloped viruses	5 min.

Active ingredient basis

Universally applicable, quick-acting, aqueous-based solution ready for use with amines, QAC, alcohol and tensides.

Efficacy/Certification

- **bactericidal** incl. TBC
- **yeasticidal**
- **virucidal** active against all enveloped incl. „blood-borne“ viruses such as HBV, HCV, HIV and influenza
- certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology), meets the DGHM requirements and EN standards for the disinfection of instruments

Durability in use

Durability in use 2 days (with normal contamination) respectively when cloudy.

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Standzeit 2 Tage.

 Liquid concentrate for the cleaning of impression trays, mixing spatulas and instruments.
 Active solution can also be used in ultrasonic devices.

Active ingredient basis

Highly effective, pH-neutral cleaning concentrate for the dissolving of organic and inorganic residues, based on non-ionic and anionic tensides and complexing agents.

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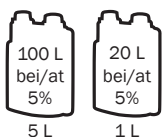
Note:

AlproSol is also suitable for the removal of alginate adhesive varnishes.

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- AlproSol** REF 3070 1 L Dosierflasche/dosage bottle*
- AlproSol** REF 3072 5 L Kanister/can*
- AlproSol** REF 3071 5 L Cubitainer*

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 *Liquid concentrate for the cleaning and self-acting removal of acid-soluble deposits (e. g. tartar and cement residues) from medical devices such as dental prostheses and instruments.*

Active ingredient basis

Acidic high-performance concentrate for the cleaning and passivation alloyed medical steels based on phosphoric acid and derivatives, tensides and polyols, ideal for cleaning of prostheses.

- with corrosion inhibitor
- also suitable on aluminium

Application

Mix 10 – 20 % with water. Application of 5 % in ultrasonic devices and in case of low soiling. Rinse the cleaned object thoroughly with water afterwards.

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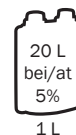
 Solution ready for use for the cleaning of the sterilizing chamber, instrument trays and steel surfaces.

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Field of application

- for the removal of residues of blood, disinfectant residues (instrument disinfection) and tissue residues
- removes powerfully in combination with SteriSponge all residues, which have formed during the sterilization process, even if they are old or heavily burned into material
- cleans powerfully with best material compatibility (as it is free from grinding parts)

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Active ingredient basis

Quick-acting, alkaline solution for the support of the mechanical cleaning based on alcohol and tensides.

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Wirksamkeit

- bakterizid
- levurozid

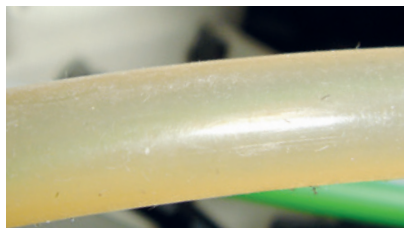
 Liquid concentrate for the continuous maintenance of the water quality in procedural water lines (including instruments and tumbler filler) of medical and dental units.
Approved by notable dental unit manufacturers.

Active ingredient basis

Disinfection concentrate for the conditioning of the procedural water in medical units based on biguanides, complexing agents and tosylchloramide sodium.

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 Solution ready for use for the decontamination and prevention of biofilm formation in procedural water lines of medical and dental units.

Application of Bilpron at least once to twice a week in the Weekend-System II or ALPRO-BCS as well as other autarkic medical water supply systems removes effectively organic and inorganic residues in procedural water lines (including the instruments and the tumbler filler).

The contact time for the disinfection solution ready for use should be at least 12 hours (overnight) in the procedural water lines, it can also be used as long term disinfection (holidays) up to 3 months.

Active ingredient basis

Solution for the sanitation, preservation and cleaning of the procedural water lines of medical units based on propylene glycol, PHB-esters, biguanides and complexing agents in an aqueous solution.

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Approved for idle time/long term disinfection amongst others by Chirana-Dental, Heka Dental, Planmeca, Ritter, Dentsply Sirona, Takara Belmont, TGA, EMS, NSK, Mectron, Ultradent.

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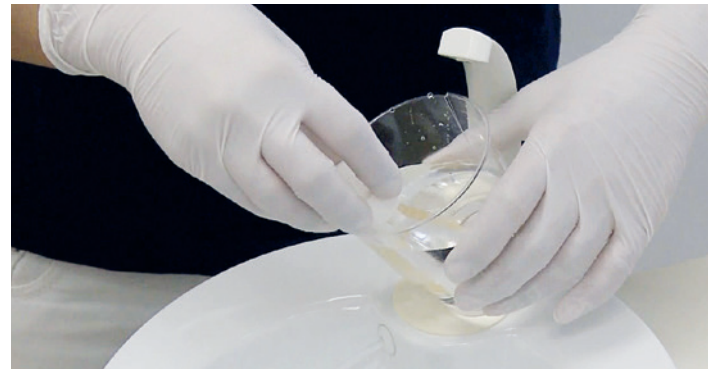
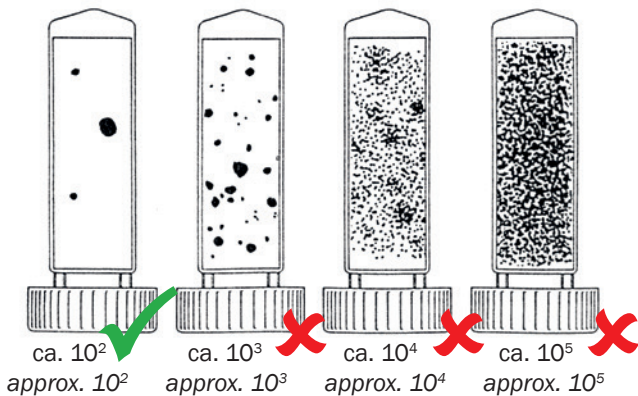
 Dip slide for semi-quantitative monitoring of microbial load of water and surfaces.

Field of application

The dip slide is used for the semi quantitative monitoring of the microbial load of liquids and surfaces in industry, environment and medicine.

The microbial count in liquids is measured in "colony-forming units" (CFU) per ml, on surfaces in CFU per cm².

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 Solution for the virucidal final disinfection of surfaces of medical devices, such as light guides, polymerisation lamps, heads of intraoral cameras/scanners, as well as for the disinfection/processing of wipe dispenser boxes and pressure bottles (1 – 1.5 L) of stand-alone procedural water supply systems such as the ALPRO-BCS (BottleCareSystem).

Active ingredient basis

Concentrate for the routine processing of ALPRO-BCS (BottleCareSystem) based on NaOCl.

Application

Pressure bottles of procedural water supply systems:

Pour the content of the bottle (100 ml) of **BC-San 100** into the empty pressure bottle and fill it with drinking water up to the thread. Leave the solution in the pressure bottle for at least 6 hours (e.g. overnight). During disinfection, keep the filled pressure bottle closed in the processing room.

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 Set for the procedural water supply, basic cleaning and disinfection of procedural water lines of medical and dental units.
Meets the requirements according to **DIN 1988-100/EN 1717** and provides an easy hygiene management of the dental unit water lines and the connected dental unit.

Field of application

- meets the standards DIN 1988-100 and EN 1717
- each dental unit can be individually upgraded
- processed tap water (with Alpron) or drinking water can be used for the treatment
- the disinfection of the water lines with Bilpron is carried out weekly in idle times (at least for 12 hours) or during vacations

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 Chair-side device for feeding the procedural water lines of medical and dental units with Bilpron.

Field of application

- easy handling
- high degree of safety and longevity
- can be connected to every dental unit without an own decontamination system
- the disinfection of the water lines with Bilpron is carried out weekly (at least for 12 hours) or during vacations

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- M.Suchomel u.a. Testing of the WHO recommended formulations in their application as hygienic hand rubs, EN 1500 Am J Infect Control 2011; 40: 328-31
- M.Suchomel u.a. Testing of the WHO recommended formulations for surgical hand preparation. J hosp Infect 2011; 79: 115-8
- WHO. WHO guidelines on hand hygiene in health care. First Global Patient Safety Challenge Clean Care is Safer Care, Geneva: WHO 2009.
- Kratzel et al. 2020, DOI: 10.1101/2020.03.10.986711 SARS-COV 2
- Steinmann et al, Antimicrob. Resist. Infect. Control 2013: 34 begrenzt viruzid PLUS (EN 14476: 2011 behüllte Viren + unbehüllte Adeno- und Noroviren)

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MinutenSpray-classic + MinutenWipes

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PlastiSept eco + PlastiSept eco Wipes

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